

**INSTI**<sup>®</sup>

# HIV-1/HIV-2 Antibody Test



Introducing **The World's Fastest**  
Point-of-Care Test

Easy to use, early detection<sup>1</sup>

Clear and accurate results

Enhances patient flow

Expedites linkage to care



For ordering information, please contact:



**authorized  
representative**

MEDINICS (M) Sdn Bhd  
Wisma MEDINICS,  
No. 2, jalan Astaka U8/88B,  
Seksyen U8, Bukit Jelutong,  
40150 Shah Alam, Selangor,  
Malaysia



**careline  
email**

+6019 812 3352  
+6019 816 3352  
+6012 240 3352  
sales@medinics.com




**website  
contact us**

[www.medinics.com](http://www.medinics.com)




# INSTI<sup>®</sup> HIV-1/HIV-2 Antibody Test

## Sample and pour

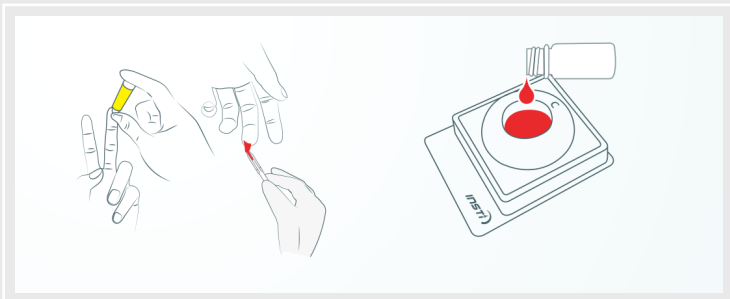

## Read result immediately!

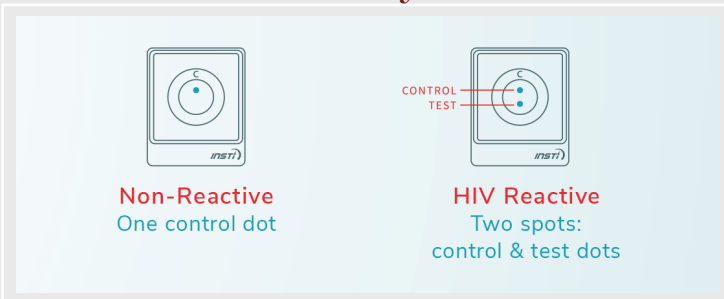

## Product information

Information Type	Product Details
Method	Flow-through
Time to results	as little as 60 seconds
Storage conditions	35.6 - 86 °F ( 2-30°C)
Shelf life	15 months
Sample type	Fingerstick, venous whole blood, plasma or serum

## Accuracy<sup>2</sup>

Sample Type	Sensitivity	Specificity
Venous Whole blood	99.4%	99.7%
Plasma	99.5%	99.96%
Fingerstick Blood	99.6%	99.3%
Serum	99%	100%

## Report your result



Scan TEST NOW QR Code to report your result. The TEST NOW platform is an online one-stop centre that provides HIV-related information including HIVST Kits as well as prevention, treatment and referral services. TEST NOW was developed in collaboration between Malaysia AIDS Foundation (MAF) and MOH. <https://www.testnow.com.my>



## INSTI HIV Self Test



## Other blood-based HIV Self Tests



## Oral Fluid HIV Self Test

Sample Type	One drop of blood	Blood	Oral fluid
Sampling Device	Not required	Requires correct use of a collection device	Requires correct use of a collection device
Early Detection <sup>1</sup>	14.9 days	20.2 days	22.9 days
Targets for Detection	IgM and IgG	IgG	IgM and IgG
Storage	2 - 30 °C	8 - 30 °C	2 - 30 °C
Timing Device	Not required	Timing device required and not provided	Timing device required and not provided
Wait time	None. Instant result	15 minutes or longer to result.	20 minutes or longer to result
Sampling Limitation	None.	Must wipe first drop of blood	Do not eat or drink for at least 15 minutes before testing. Do not use mouth cleaning products for 30 minutes before testing

No timers required

Early IgM detection<sup>1</sup>

IgG Capture Procedural Control

Lancet, pipette & alcohol swab included



**Trusted Technology That Works.**  
Confidence. The protein-A control feature serves as a built-in internal control and confirms correct sample addition, test procedure and proper test performance.

## REFERENCES

<sup>1</sup> Adams S, et al. Performance evaluation of the point-of-care INSTI<sup>®</sup> HIV- 1/2 Antibody Test in early and established HIV infections. Journal of Clinical Virology (2017) 91: 90 – 94.

<sup>2</sup> INSTI HIV-1/HIV-2 Antibody Test Package Insert (CAN), 51-1028 (J).